



SAGENT™

October 07, 2009

Sagent Pharmaceuticals, Inc.

1901 N. Roselle Road, Suite 700
Schaumburg, IL 60195
(866) 625-1618

www.SagentPharma.com

Dear Healthcare Professional

SUBJECT: CAUTION: Product-Connector Incompatibility Notice: Certain Needleless I.V. sets with Luer-Activated Devices (LADs) may be incompatible with the Sagent Adenosine and Amiodarone Prefilled Syringes.

We would like to take this opportunity to inform you of several Luer-Activated Devices (LADs) or commonly called 'connectors' with which Sagent Pharmaceuticals, Inc.'s (Sagent's) ADENOSINE Injection, USP prefilled syringes and AMIODARONE HCl Injection prefilled syringes are physically incompatible.

Some physically incompatible LADs or connectors are:

Distributor/ Manufacturer	Luer-Activated Device (LAD)
Hospira	Lifeshield® Clave™ and Microclave® Connector
ICU Medical	Clave® and Microclave® Connector
Rymed	Invision-Plus® Neutral® IV Connector
Vygon	Bionector®

As with other glass syringes, our ADENOSINE syringes are physically incompatible with these LADs due to the fact that the inner diameter of the syringes' luer tip is smaller than that required to enable these LADs to function properly. While the ADENOSINE syringes may be connected to these LADs successfully, the physical incompatibility significantly impairs or even prevents administration of the drug.

At Sagent, open communication with you is a top priority so that we can continue to deliver products with a focus on patient safety. Should you have any questions or concerns while utilizing this product, please contact Sagent Customer Service at 1-866-625-1618. Thank you for your interest in Sagent's products.

Sincerely,

Igoni Dokubo, M.D.
Medical Director

Clave and Microclave are registered trademarks of ICU Medical, Inc.; LifeShield is a trademark of Hospira, Inc. Bionector is a registered trademark of Vygon Corporation; Invision-Plus and Neutral are registered trademarks of Rymed Technologies, Inc.